



ingeniería médica

C/Alicante, 43-Mairena del Aljarafe,
E-41927 Sevilla –SPAIN-
Tel: +34 95-47 685 69, Fax +34 95-47 636 47
C.I.F. ES B-41433574




Disposable **VEIN STRIPPER** Varicostripp:


The disposable vein stripper Varicostripp is the perfect instrument on surgical treatment of varicose veins.

Technical features:

- ◆ Product with CE mark No. G2D 01 02 35808 003. (CE0123, Notified Body: TÜV SÜD Product Service)
- ◆ Each set consists of 2 cables (1 cable on model Invagination), 3 olives (4 olives with a central diabolo on cables on model invagination) and 1 puller.
- ◆ Cable, olives and puller are made up of polyamides.
- ◆ Radio-opaque cable and olives.
- ◆ Olives and puller in blue color to improve their visibility in the surgical area.
- ◆ Double bag packaging.
- ◆ Electron beam sterilization, (minimum dose of 25 kGy).
- ◆ Every single cable is traction-tested on production quality controls.
- ◆ Terminals diameter: 3,20 mm only, allowing easy insertion into the vein (cable diameter 1.50 mm).
- ◆ Cable lengths: Mod. 100 cm: 1 m (39 inches). Mod Invagination: 2.20 m. (86 inches).
- ◆ Available in two different models:
 - Model 100cm. To be used in case of exoluminal surgical technique (with olives).
 - Model Invagination. To be used in case of intraluminal surgical technique: The vein is tied up to the central diabolo of the monofilament and invaginated by pulling the cable.

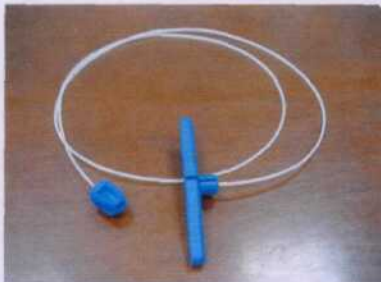


Cable (100cm), olive and handle

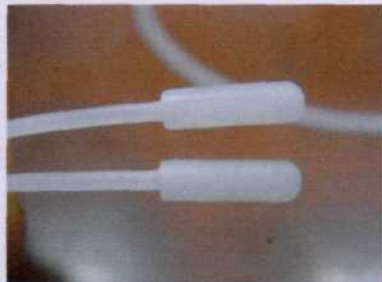


Spherical end of terminals




Box (20 sets) with opening slot

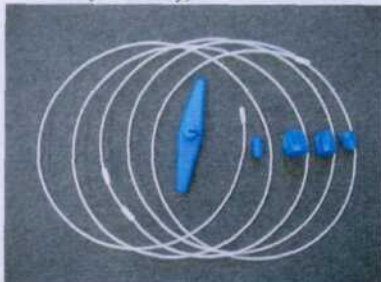

Cable 220cm, mod. Invagination

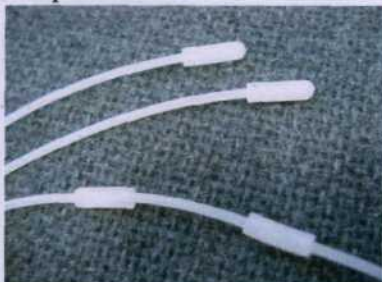

Terminals and central diabolo

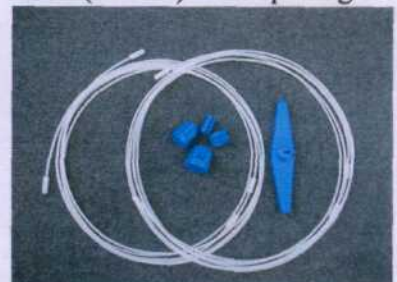

Cables, olives and puller



ingeniería médica

C/Alicante, 43-Mairena del Aljarafe,
E-41927 Sevilla –SPAIN-
Tel: +34 95-47 685 69, Fax +34 95-47 636 47
C.I.F. ES B-41433574



NEWS on Disposable VEIN STRIPPER Varicostripp:

We have changed some features (From and included LOT F040733) on our product Varicostripp to improve the product quality and as a result of customer's feedback:

- ◆ The set composition has changed: **WAS:** Tree olives: Small (9mm), Medium (13mm) and Large (15mm) plus a 6mm olive on model Invagination. **IS:** Three olives: 9mm, 11mm and 13mm plus a 6mm olive on model Invagination. The 15mm olive is lost and we add a new 11mm olive. The denomination for the olives has also changed to a number + mm corresponding to the actual diameter of the olive, and the size (small, medium..) is not used.
- ◆ The product references have changed to FLV100 instead of FL100 and FLVINV instead of FLINV.
- ◆ The expiration date / shelf life is now 5 years instead of 3 years, this means a 2 years increment.
- ◆ The stops, ends or terminals of the cables now will fit in the olives so the olive is blocked to the cable.
- ◆ The product is now labelled as LATEX FREE and we have a latex free certificate to be sent by request. There is not a standardized symbol for latex free, so we designed our own symbol. We think it is clear enough for everybody.

**LATEX
FREE**

The changes are due to the 15mm olives are close to never be used and the new 11 mm olive is a useful size.

The reference is changed to allow a better identification between the old and the new sets.

The shelf life is increased from 3 to 5 years after a follow-up of the internal and external pouches aging, the cables traction tests and set elements aging for 7 years.

The stops of the cables fit in the olive's hole (before were free) because some Doctors ask for this change.

Our product was latex free before, but now it is labelled on each pouch and box label.

We made these changes to improve the quality of our product and hope that you and your customers will find the changes as improvements.

We will appreciate any hint or suggestion from our customers and distributors about our products, as a feedback necessary to maintain and improve the product characteristics and quality.

Our best Regards: J. Zaranton, QS Manager

INGENIERIA MEDICA, S.L
Alicante nº 43, E-41927, Mairena del Aljarafe, Seville. –SPAIN-
FAX: +34-95-47 636 47. Phone: +34-95-47 685 69
e-mail: inmedica@auna.com